CANTOL

SEELECT A-M 256

DISINFECTANT-CLEANER
SANITIZER-FUNGICIDE
MILDEWSTAT-VIRUCIDE*
DEODORIZER

NETTOYANT-DESINFECTANT
ASSAINISSANT-FONGICIDE
MILDEWSTAT-VIRUCIDE*
DESODORISANT

for Hospitals, Institutional and Industrial Use Pour utilisation dans les hôpitaux, institutions et établissements industriels

Effective in hard water up to 400 ppm hardness (calculated as CaCO3) in the presence of 5% serum contamination. Efficace en eau dure jusqu'à 400 ppm de dureté en calcium (sous forme de CaCO3) en présence de 5% de sérum sanguin.

COMMERCIAL

WARNING / AVERTISSEMENT POISON / CORROSIVE / CORROSIF

GUARANTEE / GARANTIE:

Alkyl (40% C12, 50% C14, 10% C16) dimethyl	benzyl
ammonium chloride	8.680%
Octyl decyl dimethyl ammonium chloride	6.510%
Dioctyl dimethyl ammonium chloride	2.604%
Didecyl dimethyl ammonium chloride	3.906%

READ THE LABEL BEFORE USING LIRE L'ETIQUETTE AVANT L'EMPLOI

REGISTRATION NO. 22229
PEST CONTROL PRODUCTS ACT
NO. D'ENREGISTREMENT 22229
LOI SUR LES PRODUITS ANTIPARASITAIRES

DIN 873217

NET CONTENTS / CONTENU NET: L

Manufactured by / Fabriqué par:

CANTOL LIMITED 199 STEELCASE ROAD WEST, MARKHAM, ONTARIO L3R 2M4

Cross-contamination is of major housekeeping concern not only in hospitals, but in schools, institutions, and industry. CANTOL SEELECT A-M 256 has been formulated to aid in the reduction of cross-contamination in these areas.

CANTOL SEELECT A-M 256 is a one-step disinfectant-cleaner that is effective against a broad spectrum of bacteria, is virucidal, fungicidal and will prevent the growth of mold and mildew and their odors when used as directed.

CANTOL SEELECT A-M 256 is an effective one-step sanitizer-cleaner for use on non-food contact surfaces.

When used as directed, CANTOL SEELECT A-M 256 will deodorize surfaces in toilet areas, behind and under sinks and counters, garbage cans and garbage storage areas and other places where bacterial growth can cause malodors.

PRECAUTIONARY STATEMENTS:

KEEP OUT OF REACH OF CHILDREN.

Danger - Corrosive to eyes. Do not get in eyes, on skin or on clothing. Harmful if swallowed. Avoid contamination of food. FIRST-AID: In case of contact, immediately flush eyes or skin with plenty of water for at least 15 minutes. For eyes, call a physician. Remove and wash contaminated clothing before reuse. If swallowed, drink promptly a large quantity of milk, egg whites, gelatin solution, or if these are not available, drink large quantities of water. Avoid alcohol. Call a physician or poison control centre immediately.

TOXICOLOGICAL INFORMATION: Probable mucosal damage may contraindicate the use of gastric lavage. Measures against circulatory shock, respiratory depression and convulsion may be needed.

DISPOSAL: 1. Rinse the emptied container thoroughly and add the rinsings to treatment site. 2. Follow provincial instructions for any required additional cleaning of the container prior to its disposal or reconditioning. 3. Dispose of the container in

accordance with provincial requirements. 4. For information on the disposal of unused, unwanted product and the cleanup of spills contact the Provincial Regulatory Agency or the Manufacturer. 5. For small spills. Recover free liquid. Soak up residue with solids absorbent. DO NOT FLUSH INTO SURFACE STREAMS. INFORM ENVIRONMENT CANADA AS IN NO. 4.

NOTICE TO USER:

This control product is to be used only in accordance with the directions on this label. It is an offense under the Pest Control Products Act to use a control product under unsafe conditions.

DIRECTIONS FOR USE:

CANTOL SEELECT A-M 256 is a proven "one-step" disinfectant-cleaner-sanitizer-fungicide-mildewstat-virucide which is effective in water up to 400 ppm hardness in the presence of 5% serum contamination.

Apply CANTOL SEELECT A-M 256 to walls, floors and other hard (inanimate) non-porous surfaces such as tables, chairs, countertops, sinks, tile, porcelain, and bedframes with a cloth mop so as to thoroughly wet surfaces. For heavily soiled areas, a preliminary cleaning is required. Prepare a fresh solution daily or when use solution becomes visibly dirty.

DISINFECTION: To disinfect hard, non-porous surfaces, add 4 mL/L of water. Treated surfaces must remain wet for 10 minutes. At this use level. CANTOL SEELECT A-M 256 is also virucidal and fungicidal.

SANITIZING: To sanitize non-porous, non-food contact surfaces, add 1 mL/L of water. Treated surfaces must remain wet for 60 seconds.

4 mL/L Use-Level: The broad spectrum activity of CANTOL SEELECT A-M 256 has been evaluated and found to be effective against the following organisms by the AOAC Use-Dilution test:

Pseudomonas aeruginosa
Salmonella choleraesuis
Streptococcus pyogenes
Enterobacter aerogenes
Streptococcus faecalis
Brevibacterium ammoniagenes

Staphylococcus aureus Escherichia coli Klebsiella pneumoniae Salmonella schottmuelleri Shigella dysenteriae Salmonella typhi AOAC Fungicidal Test: CANTOL SEELECT A-M 256 is an effective fungicide against Trichophyton mentagrophytes (the athlete's foot fungus) when used on surfaces in areas such as locker rooms, dressing rooms, shower and bath areas, exercise facilities, etc.

Mold and Mildew Control: at 4 mL/L CANTOL SEELECT A-M 256 will effectively inhibit the growth of mold and mildew and the odors caused by them when applied to hard, non-porous surfaces (as indicated in general cleaning instructions above). Allow to dry on surface and repeat when mildew growth returns.

Virucidal Performance*: at 4 mL/L use-level, CANTOL SEELECT A-M 256 was evaluated and found to be effective against the Influenza A/Brazil Virus on inanimate environmental surfaces.

Sanitizing: Non-Food Contact Surfaces (such as floors, walls, tables, etc.): at 1 mL/L use-level, CANTOL SEELECT A-M 256 is an effective sanitizer against Staphylococcus aureus and Klebsiella pneumoniae on hard non-porous environmental surfaces.

This label transcript service is offered by the Pest Management Regulatory Agency to provide efficient searching for label information. This service and this information do not replace the official hard-copy label. The PMRA does not provide any guarantee or assurance that the information obtained through this service is accurate, current or correct, and is therefore not liable for any loss resulting, directly or indirectly, from reliance upon this service.

+))